

Classroom Positioner Desk/Stander Instructions for Use

1. **TRAY HEIGHT ADJUSTMENT;** To change the height of the tray loosen knobs **A** on both sides of the desk and raise or lower the tray to the desired height.
2. **TRAY ANGLE ADJUSTMENT;** To adjust the angle of the tray loosen knob **B**.
3. **ADJUST TRAY CLOSER / FARTHER:** To move the tray in or out there are two knobs **C** that must be loosened.
4. **KNEE BLOCK POSITION:** To move the position of the knee block there are four knobs **D** that must be loosened

--AN EXPERIENCED THERAPIST OR REHAB TECHNOLOGY PROVIDER SHOULD ALWAYS BE CONSULTED WHEN MAKING ANY FITTING ADJUSTMENTS

--THE DESK/STANDER MUST ALWAYS BE LATCHED TO THE CLASSROOM POSITIONER CHAIR WHENEVER THE UNIT IS BEING USED AS A STANDER.

--NEVER LEAVE A CHILD UNATTENDED WHILE USING THIS EQUIPMENT

