


## Large Supine Stander Assembly Instructions

After unpacking the cartons you should have the following parts to your supine stander; main supine assembly, base assembly, tray, and headrest. (figure 1)

Figure 1

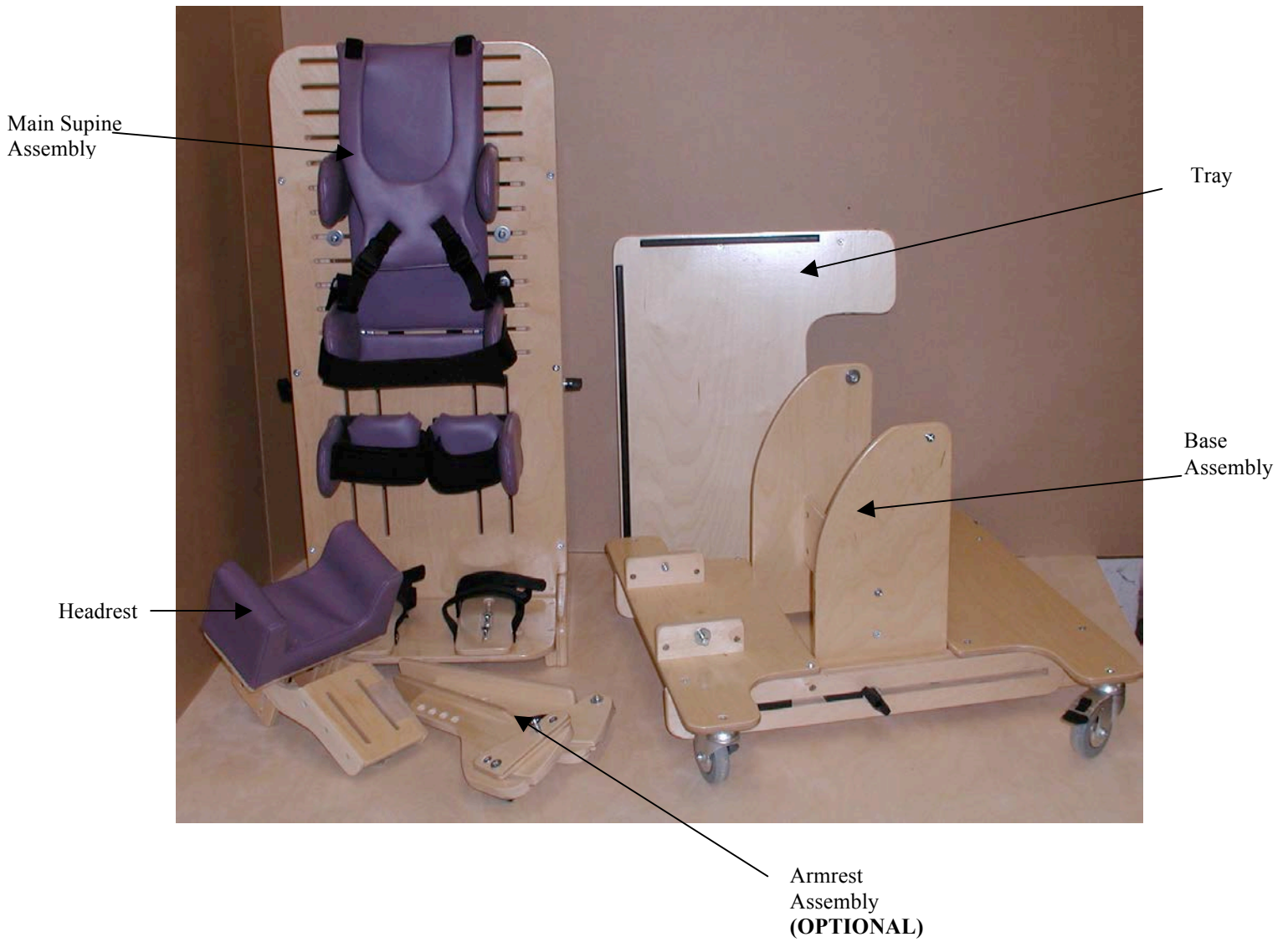

1. On the main supine assembly locate and loosen the *height adjustment knobs (A)*. Slide the supine board up to its' maximum height and tighten the knobs again.
2. Locate the *main pivot bolts (B)* located on the base uprights. Remove the bolts, nuts, and washers. Place the main supine assembly onto the base assembly so that the *main pivot bolt holes (C)* line up with the base upright pivot holes. (See figure 3)
3. Screw the pivot bolts into *main pivot bolt holes (C)* and tighten as firmly as possible. Replace the washers and nuts on the inside of base upright pivot bolt holes. Tighten the nuts and then back off ¼ turn Lower the supine board back to its' lowest position..
4. Extend the back legs so that they can reach the *back leg pivot bolts (D)*
5. Remove the nuts and washers from the *back leg pivot bolts(D)*. Gently pull the end of the back legs over the pivot bolts. Tighten the nuts and then back off ¼ turn.



Figure 2

6. Screw the pivot bolts into *main pivot bolt holes (C)* and tighten as firmly as possible. Replace the washers and nuts on the inside of base upright pivot bolt holes. Tighten the nuts and then back off  $\frac{1}{4}$  turn.
7. Extend the back legs so that they can reach the *back leg pivot bolts (D)* (figure 4)
8. Remove the nuts and washers from the *back leg pivot bolts (D)*. Gently pull the end of the back legs over the pivot bolts. Replace the washers and nuts. Tighten the nuts and then back off  $\frac{1}{4}$  turn.
9. Armrests (OPTIONAL)—The armrests will be mounted in the series of holes on the side of the main supine assembly. (figure 5) With the supine board raised all the way remove the four mounting bolts from the armrests. Estimate the desired height of the armrests and bolt in place as shown in the picture.
10. Headrest—Remove the knobs from the bag taped to the headrest and use them to mount the headrest to the main supine board as shown in figure 6.



Figure 3



Figure 4

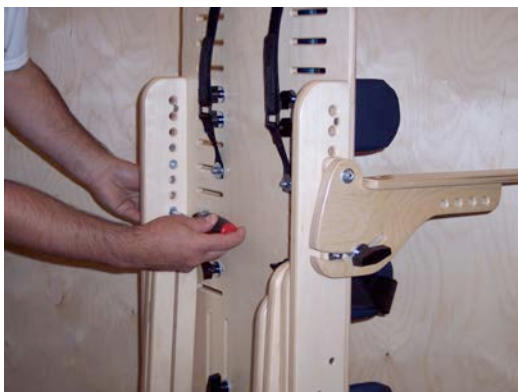


Figure 5



Figure 6